STERILE EO

Emergency delivery kit

This article is intended for exclusive use by healthcare professionals. The device must be used in accordance with the procedures of Good Clinical Practice, in conformity with the protocols and guidelines.

Detailed composition is present in labelling

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
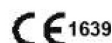
UMBILICAL CORD CLAMP

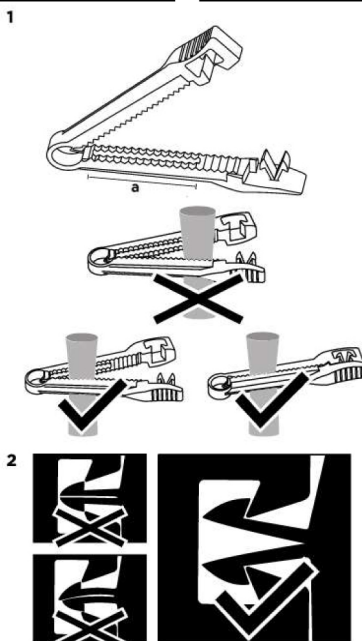CLAMP PARA CORDÃO UMBILICAL

INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

Umbilical cord clamp is intended to clamp the umbilical cord after delivery to prevent bleeding from the cord stump after the cord had been cut.

INSTRUCTIONS FOR USE

- 1- Check the thickness of the umbilical cord. In case it is a thin cord, it may be necessary to fold it before placing the clamp to obtain a bigger resistance to the clamp closing.
- 2- Place the cord within the clamp, in the clamping zone (figure 1a).
- 3- Check that there is not any object blocking the locking site (e.g. swabs, surgical instruments, etc.).
- 4- Close the clamp until it locks and clicks (figure 2). Make sure that the clamp is well closed and that it is impossible to open it with the hands.
- 5- Confirm that there is no bleeding at the end of the cord. Check regularly for signs of bleeding due to a misplaced clamp or due to a damaged cord.

Umbilical cord clamp is intended to clamp the umbilical cord after delivery to prevent bleeding from the cord stump after the cord had been cut.

WARNINGS AND PRECAUTIONS

- Intended to be used exclusively by healthcare professionals.
- Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
- Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
- Always check the integrity of the locking system before use. Do not use if incomplete or blocked. Incorrect placement of the clamp on the umbilical cord may lead to an inadequate seal and consequent blood loss that might prove fatal for some babies.
- Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.

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• When used on healthcare facilities and if contaminated: Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution. Otherwise: Dispose according to applicable legislation and/ or infection prevention standards for urban waste.

The occurrence of serious incidents is not expected with the use of the device if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your

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CONTRAINDICATIONS AND/ OR PRECAUTIONS

Store in a dry place at room temperature and protected from sunlight.

Do not use the contents if the package is opened or damaged. Do not re-sterilize. Do not reuse.

There are no other known contraindications and/or warnings, in addition to the above, specific for the component(s) in question.

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