Rescue sheet

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FR Couverture de secours

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NL Isolatiedeken

IT Coperta d'emergenza

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EN RESCUE SHEET

INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

Rescue sheet (Silver): Ag SILVER

To cover patients to maintain the normothermia of the body, preventing hypothermia.

Rescue sheet (Silver/Gold): Ag/AU GOLD

To cover patients to maintain the normothermia of the body, preventing hypothermia (when used with golden side up) or preventing hyperthermia (when used with silver side up).

INSTRUCTIONS FOR USE

1- Open the package, remove and unfold the rescue sheet. Put the golden side up to protect from cold or the silver side up to protect from heat.
2- If the person is injured and/or unconscious, place them properly and in a safe position.
3- Wrap the rescue sheet around the person, avoiding covering the face.
4- Keep the person covered until evaluated by a healthcare professional.

DEVICE INFORMATION

- Rescue sheet is intended to be used by healthcare professionals or lay persons.
- Resistant, waterproof and windproof.
- Lightweight.
- Reflective material allows high visibility, being adequate to be used as a signalling device.

WARNINGS AND PRECAUTIONS

- Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
- Rescue sheet is intended for personal and single use.
- Do not cover the face with the rescue sheet.
- Do not tear or puncture the rescue sheet.
- When used on healthcare facilities, and if contaminated: Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.
Otherwise: Dispose according to applicable legislation and/or infection prevention standards for urban waste.

The occurrence of serious incidents is not expected with the use of the device correctly, but according to Regulation (EU) 2017/745 we must inform the users of the risks related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

LABELLING SYMBOLS

1)Opening direction 2)Paper (Identification of packaging material)
3)Swiss authorized representative

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