## EN KOCHER FORCEPS

### INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

Kocher forceps is intended to be used for:
-Grasping skin and soft tissues in woundcare procedures.
-Holding swabs for absorption of blood and fluids during surgery or woundcare procedures.
-Holding swabs for skin disinfection procedures.

### WARNINGS AND PRECAUTIONS

- Intended to be used exclusively by healthcare professionals.
- Kocher forceps is not intended to be used for hemostasis or to clamp vessels.
- Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
- Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
- Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.
-Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.

### INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

The occurrence of serious incidents is not expected with the use of the device if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

### INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

### INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

### INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

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