## EN Personal Protective Equipment Regulation (EU) 2016/425

### Personal Protective Equipment Regulation (EU) 2016/425 as brought into UK law and amended

#### INTENDED PURPOSE

Particle filtering half mask is intended to protect against both solid and liquid aerosols. Particle filtering half masks are classified according to their filtering efficiency and their maximum total inward leakage. There are three classes of devices: FFP1, FFP2 and FFP3. The protection provided by an FFP2- or FFP3- device includes that provided by the device of lower class or classes.

Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

#### FITTING INSTRUCTIONS

- Place fingers inside the respirator. Bend the nose clip around fingers to form nose shape.
- Hold the mask in position over the nose and mouth, with the retaining straps around the back of the head, and pull the lower portion of strap over head to fit around the neck.
- Stretch and pull the upper strap over your head and adjust it on the back of your head above your ears. The straps must not be twisted.
- Using both hands, place the clip around the nose and adjust the tension in the upper and lower portions of the continuous loop face strap, in order to achieve a proper face seal.
- The seal of the respirator on the face should be fit-checked before entering the workplace. To fit check:
  - UNVALVED: Place both hands over the respirator and exhale vigorously.
  - VALVED: Place both hands over the respirator and inhale sharply. A negative pressure should be felt inside the respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the head strap for a better fit.
- Change respirators immediately if breathing becomes difficult or if respirator becomes damaged or distorted, or a proper face fit cannot be maintained.

End of shelf life (yyyy/mm)	See information supplied by the manufacturer	EN149:2001+A1:2009 465-102 FFP2 NR, non-valved
Temperature range of storage conditions (-30°C - 40°C)	Manufacturer	⊗ / NR = Non-reusable (single shift only)
Maximum relative humidity of storage conditions (<80%)	Manufacturer	

#### WARNINGS AND PRECAUTIONS

- A properly selected respirator is essential to protect your health. When in doubt, ask a healthcare or safety professional before using.
- Do not use against gases/vapours.
- Do not use this respirator when the oxygen concentration is less than 19%.
- Do not use in explosive environments.
- Do not use when concentrations of contaminants are unknown or immediately dangerous to health or life.
- This product does not supply oxygen. Use only in properly ventilated areas containing enough oxygen to support life.
- Do not use with beards or certain facial features that prevent an effective seal between mask and face.
- Leave work area immediately and return to fresh air if breathing becomes difficult, or if there is dizziness or other distress.
- Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of the respirator and result in sickness or death.
- Do not re-use. Particle filtering half mask is intended to be used one single time to prevent risks from contamination and/or functional damage.
- Do not use the device if you detect any sign of deterioration that may compromise its use and/or functionality.
- When used on healthcare facilities, and if contaminated, dispose according to local legal regulations for healthcare waste. Otherwise, dispose according to local legal regulations for urban waste.

The occurrence of serious incidents is not expected with the use of particle filtering half mask, but we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country. \*Internet address where the declaration of conformity can be accessed.

CE 0598  
SGS TIKMOK OY  
TAMKOLA 8 00380 HELSINKI Finland

SGS United Kingdom Limited  
Rossmore Business Park, Elmsmere Port, South Wirral, Cheshire, CH65 3EN, United Kingdom

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